Twée Go

Breast pump

Laktator

Milchpumpe

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Tiralatte

Tire-lait

Extractor de leche

Borstkolf

Krūties siurblys

Odsávačka

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Mellszívó

Pompa de san

Bröstpump

Brystpumpe

Brystpumpe

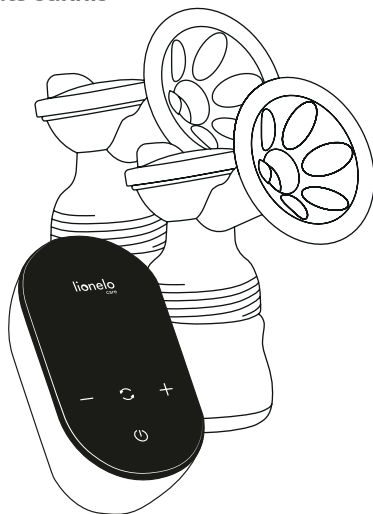
Rintapumppu

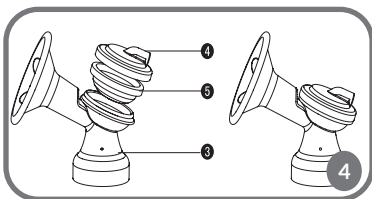
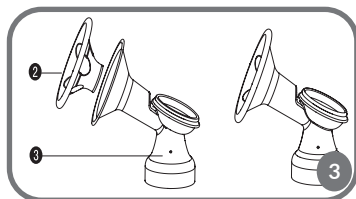
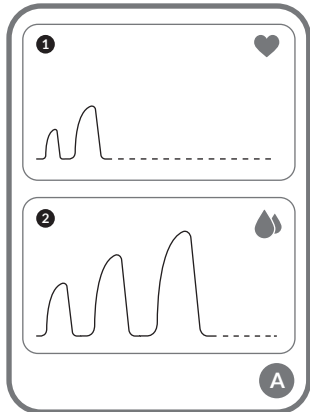
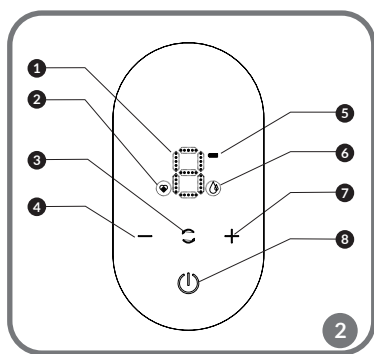
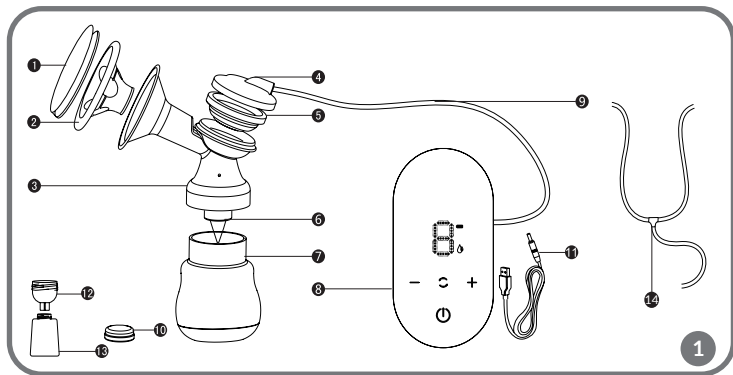
Bomba tira leite

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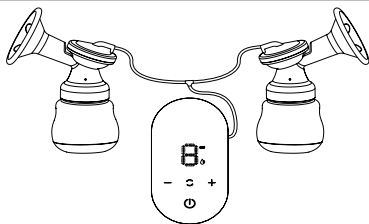
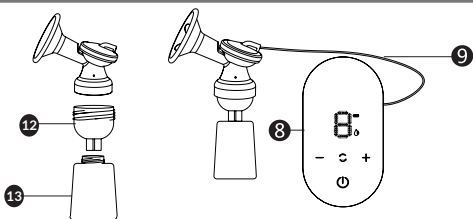
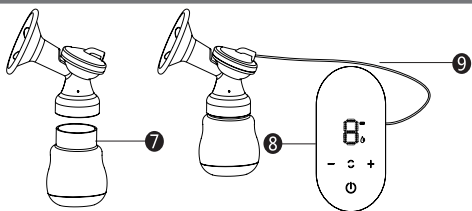
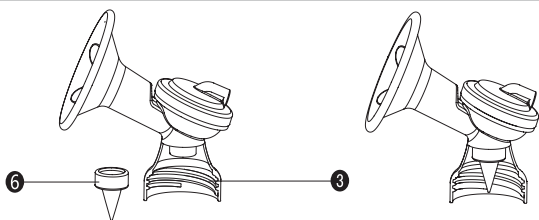
Rinnarpump

Krūts sūkņis

IMPORTANT

Keep this manual for future reference.

1. Do not repair or modify the device. Only an authorized service technician is allowed to perform these operations.
2. This device is not a toy. Do not let children play with it. Keep the device out of reach of children. Cleaning and maintenance must not be performed by children if unattended.
3. The device may be used by children aged at least 8 and persons with limited physical, sensory or mental capabilities, as well as persons who lack relevant knowledge and experience in using the device as long as their actions are supervised or they have been instructed on safe use of the device and related risks beforehand.
4. Do not start the device if you have noticed any damage. This applies to the device and its accessories, e.g. USB cable.
5. Do not use the device when it starts acting up, has been dropped or immersed in water, overheats extensively, is discolored or deformed, and when other atypical circumstances occur. Contact the manufacturer's authorized service center.
6. Never leave the plugged device unattended. The device can only be charged with a SELV charger with the following output parameters: 5V DC 2A.
7. Use the device away from sources of heat, high temperatures, hot surfaces, direct sunlight, sources

- of ignition, naked flame, oils and sharp edges.
- 8.** Do not use chemicals to clean the device. Do not wash the main body and motor with water.
 - 9.** Switch the device off after every use.
 - 10.** Do not immerse the entire device in water or other liquids. Do not use near water.
 - 11.** Do not disinfect the device with a steam sterilizer.
 - 12.** Do not use the device at variance with intended use.
 - 13.** Do not use the device and do not touch the power cable if your hands are wet.
 - 14.** Before cleaning or moving the device, switch it off and unplug.
 - 15.** Before and after use, wash and disinfect all pieces which will adhere to the body and come into direct contact with milk.
 - 16.** Carefully dry every piece before use.
 - 17.** Do not use a tube which links the main motor with breast pump T-connection unless you have applied the silicone membrane beforehand. The tube does not need to be disinfected.
 - 18.** For hygiene-related reasons, the device must be used by one person.
 - 19.** If use of the breast pump causes discomfort or pain, stop using the device immediately.
 - 20.** Do not use the device throughout pregnancy.
 - 21.** Do not use the device if you are tired or if you are sleeping.
 - 22.** If milk does not come out of the breast for more than 5 minutes while you are using the product, stop using the breast pump and contact your doctor.

The product complies with requirements of EU directives.

Pursuant to the Directive 2012/19/EU, this product is subject to selective collection.

The product must not be disposed of along with domestic waste since it may pose a threat to the environment and human health. Worn-out product must be handed over to the electrics and electronics recycling point.

Device outline (fig. 1)

- | | |
|-----------------------------|----------------------|
| 1. Protective cover | 8. Main motor |
| 2. Silicone funnel | 9. Tube |
| 3. Breast pump T-connection | 10. Milk bottle cap |
| 4. Membrane cover | 11. USB cable |
| 5. Silicone membrane | 12. Milk bag adapter |
| 6. Suction valve | 13. Milk bag |
| 7. Milk bottle | 14. Connector |

Touch screen outline (fig. 2)

- | | |
|--------------------------------|----------------------|
| 1. Massage and suction display | 5. Battery indicator |
| 2. Milk stimulation mode | 6. Milk drawing mode |
| 3. Mode switching | 7. Level up |
| 4. Level down | 8. On/off switch |

Breast pump modes (fig. A)

1. Milk stimulation mode – 5 levels. Massage stimulates initial child suction phase while breastfeeding.
2. Milk drawing mode – 9 levels. This mode stimulates deeper child suction phase while breastfeeding.

How to install

1. Position the silicone funnel (2) on the breast pump T-connection (3). Make sure the funnel flange is properly attached (fig. 3).
2. Place the silicone membrane (5) on the breast pump T-connection (3). Make sure the membrane flange is properly attached (fig. 4), and then put the membrane cover (4) on.
3. Insert the suction valve (6) into the lower hole of the breast pump T-connection (3). Make sure the valve is pushed as fast as possible. Note: insert the suction valve gently to avoid damage. Do not use the valve (fig. 5) if it is damaged.
4. Screw the milk bottle (7) to the breast pump T-connection (3). Insert the tube (9) into the hole in the membrane cover. Connect the membrane cover

to the main motor (8) with the use of the tube (fig. 6). Make sure the tube is tight to prevent air from escaping.

5. If you want to keep milk for future use, screw the adapter (12) to the milk bag (13), and then screw everything to the breast pump T-connection and connect T-connection to the main motor (8) with the use of the tube (9) (fig. 7).
6. To connect the double breast pump, retake the aforesaid installation activities with the second set and connect tubes to the main device with the use of the connector (fig. 8).

How to use

NOTE

Before use, make sure the breast pump has been properly installed and all its pieces which come into contact with milk are disinfected. Wash hands. While using, press the breast pump against the breast tightly to avoid air suction.

1. Switch the device on, short press the on/off switch. The device will activate milk stimulation mode (massage) automatically.
2. The mode ends automatically after 3 minutes. You can also change the mode manually before 3 minutes expires. To do so, press the mode switching key (fig. 2).
3. The milk stimulation mode: 5 levels are available. You can change massage level by pressing level up or down keys (fig. 2).
4. Milk drawing mode: 9 levels are available. You can change the suction level by pressing level up or down keys (fig. 2).

Selection of a suitable and comfortable level is very individual and depends on the suction power. It is advisable to use the breast pump in the milk drawing mode directly after the milk stimulation mode.

The device deactivates automatically after a default working time: 30 minutes.

How to store and heat up milk

How to store

1. After collecting milk, tightly close the container and put inside the fridge.
2. Milk can be stored for up to 4 hours in the room temperature (16-29°C).
3. Milk can be stored for up to 48 hours in the fridge (4°C).
4. Milk can be stored for up to 3 months in the freezer (< -4°C).

How to heat up

After defrosting or taking out of the fridge, use the heater to heat the milk and serve for feeding. Do not use a microwave or boiling water to heat it up because milk may lose its nutritional properties.

NOTE

1. To freeze milk, make sure $\frac{3}{4}$ of the container is filled because frozen milk expands.
2. To maintain nutritional values of the milk while defrosting, at first place the milk in the fridge, and then heat up with warm water at the maximum temperature of 37°C.
3. Do not use microwave or boiling water to defrost milk to avoid loss of nutritional properties.
4. Never try to freeze milk again if it has already been defrosted.
5. Never add fresh milk to frozen milk.
6. Before you start breastfeeding, gently shake the milk container to mix it properly.

How to charge

1. The breast pump has a built-in battery that can only be replaced by a qualified person.
2. When the battery is low, battery symbol will show one bar. In such case it is advisable to charge the device.
3. Connect USB cable to breast pump USB power supply and the other tip to the USB power adapter (not supplied).
4. When the battery is full, battery symbol glows and shows three bars.
5. You can use the breast pump while charging the battery, but please remain very careful.

Troubleshooting

Problem	Solution
Device does not work	• Check battery status • Start charging the device • Make sure the device has been correctly connected to charging source
Insufficient suction power	• Make sure the suction valve is correctly installed • Place the valve in boiling water, leave for 3 minutes and then fix again • Make sure battery level is sufficient • Make sure the breast pump adheres to the breast tightly • When temperature in the room is lower than 15°C, leave the working device for about 30 seconds before you start using it.
Pain or discomfort	• Adjust massage/suction level • Change position or location of the breast • In case the pain or discomfort does not subside, contact your doctor

How to clean and disinfect

1. After use, disassemble the breast pump and clean the main device with a dry and soft cloth.

2. Before first use, disinfect all pieces which will come into contact with milk (milk bottle, T-connection, valve, membrane and silicone funnel).
3. To disinfect, put all pieces into boiling water for 10 minutes. Make sure all pieces are immersed in water and none of them touches a dry part of the vessel, because these pieces may melt.
4. Before another use, wash all pieces under running water in order to remove food remains.
5. The tube must be completely dry before use – moisture may be sucked into the main device and cause damage.
6. Leave all pieces on a clean cloth or dish drainer until they go dry completely.

NOTE

The main device and USB cable must not be disinfected and cleaned with water. Disinfection applies only to accessories which come into contact with milk.

When cleaning, be careful because damaged piece may cause the breast pump to malfunction.

The pictures are for reference only. The real design of the products may differ from the pictures presented.

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