**RIDDER**

*Assistent*

★★★★★ PREMIUM

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**RIDDER GmbH**

Ridder Straße 1, D-56379 Singhofen

Tel.: +49 (2604) 9541-0, Fax +49 (2604) 9541-50

Mail: [info@ridder-online.de](mailto:info@ridder-online.de), [www.ridder-online.de](http://www.ridder-online.de)

Art.-Nr.: A0300100 1352018

DE

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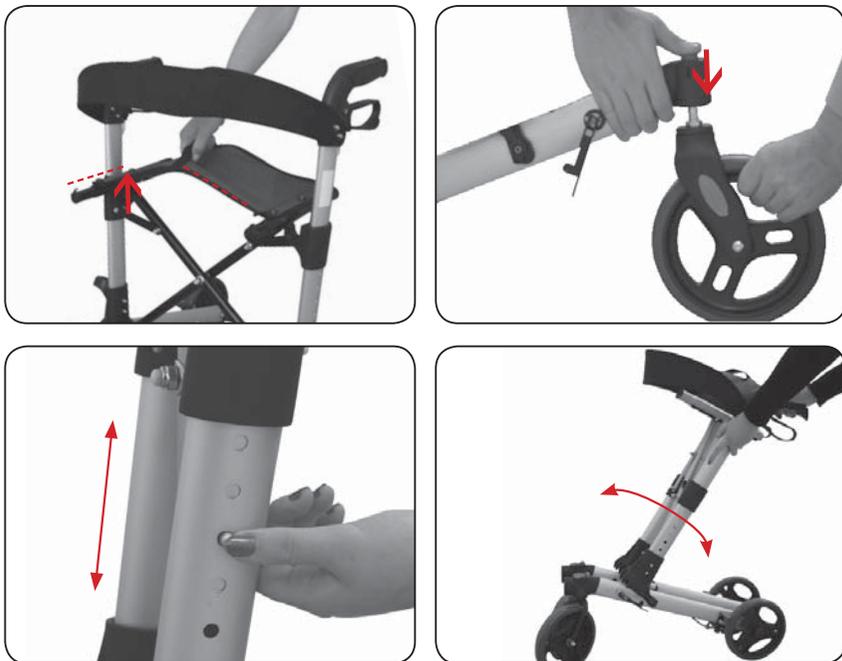
GB


**USER MANUAL**

Fold

Usage of the brakes

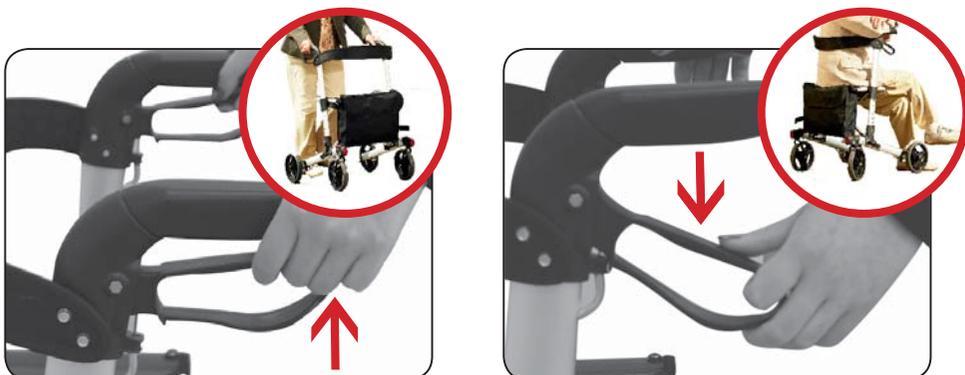**! WARNING!**

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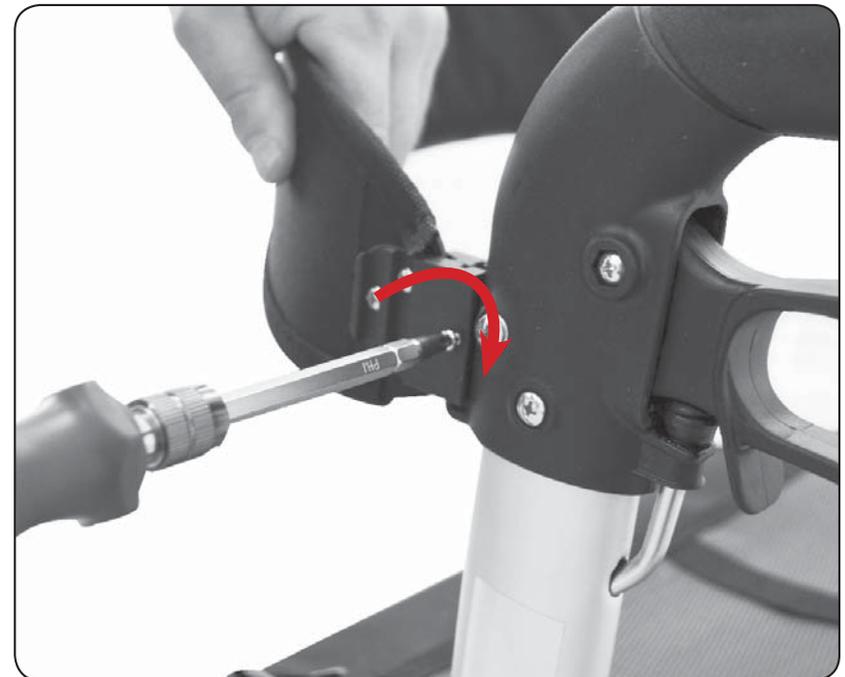
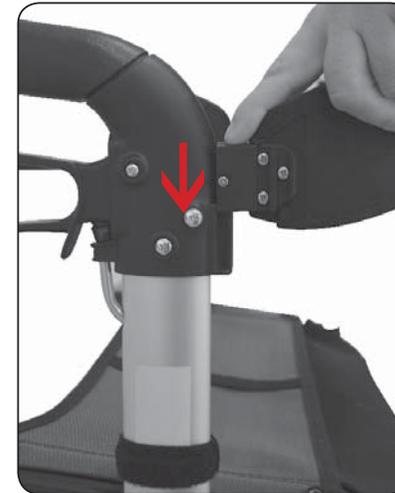
Risk of injury by hand trapping!

**i NOTE!**

**Important:** Check all parts of shipping damage. In case of shipping damage, DO NOT use the product, contact the distributor for further instructions.

**i NOTE!**

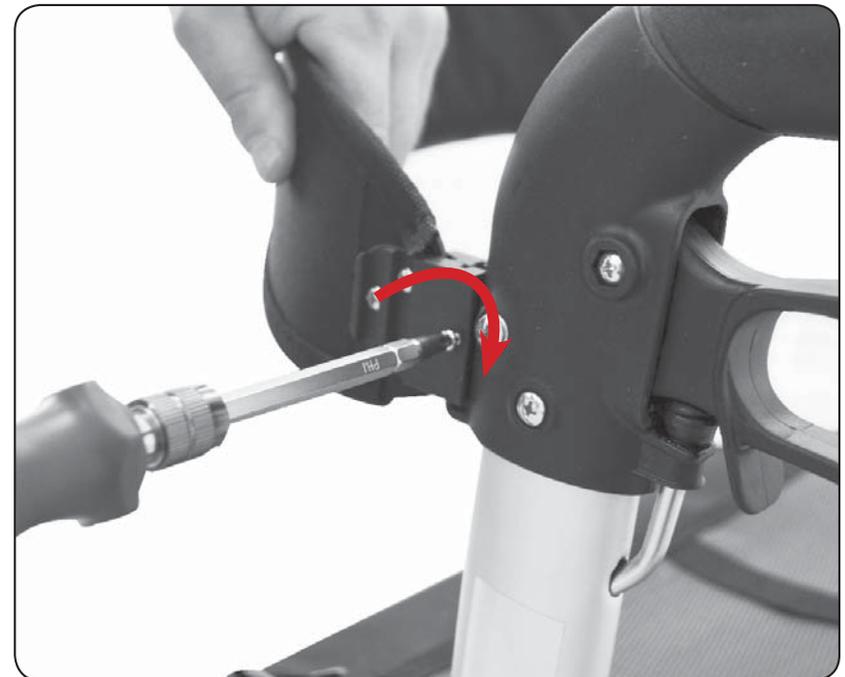
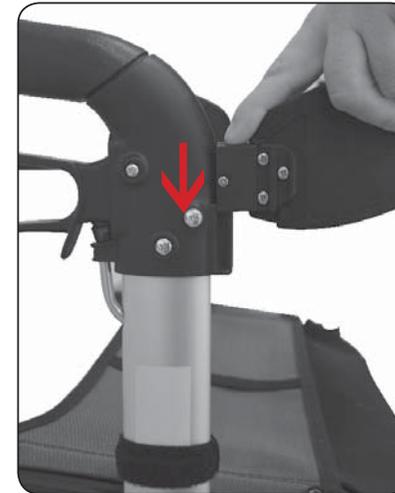
**Stability warnings:**

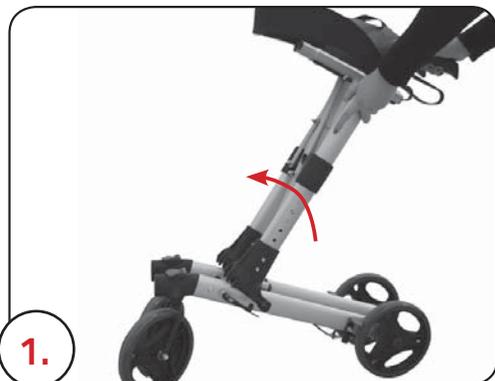
The walking frame can provide ambulatory assistance to an individual weighing up to 136kg (300lbs), INCLUDING the loading of the bag. The bag has a weight limitation of 5kg (11lbs).

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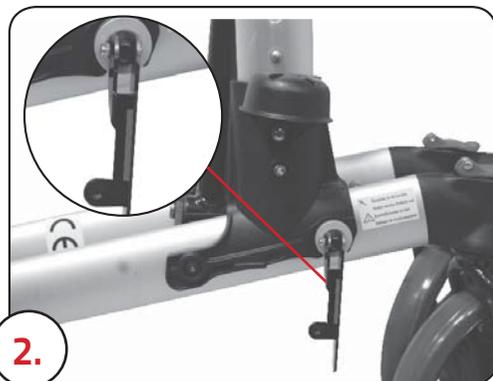
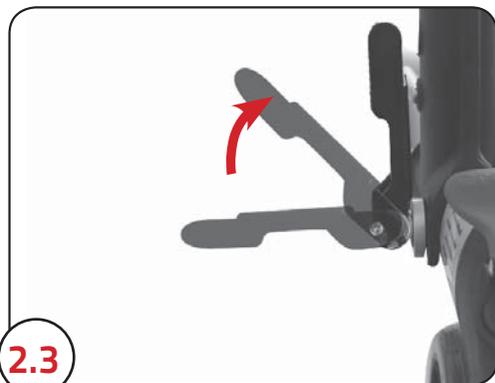
2.4

## **i** NOTE!

Failure to follow these warnings can result in personal injury. DO NOT install or use this equipment before reading and understanding this instruction sheet. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional or technical personnel before attempting to install this equipment – otherwise, injury or damage may occur. You should always consult with your physician or therapist to determine the proper adjustment and usage.

The walking frame, which is usable in- and outside, was developed as a walking aid for motor-impaired humans. It can be used as seating as well.

The maximum recommended useful life is up to 5 years. In this specification the consideration of the intended use-, safety- and care instructions is compulsory prerequisite.

## **!** WARNING!

- Walking frames are NOT intended to be self-propelled while seated.
- Always check if the seat tube is locked and well connected to the frame when the Walking frame is open.
- Always test the brake after every height adjustment in order to avoid a free flex of the cable.
- Always test, whether the both of the handgrip rails are secured at the same height and the snap buttons for the handgrip height adjust are fixed correctly (are out)
- All wheels MUST be in contact with the floor at ALL TIMES during use. This will ensure the Walking frame is properly balanced.
- Do not use in case of damage.

## **i** NOTE!

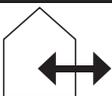
### Cleaning & Care:




Please follow the cleaning and care instructions. Improper use may damage the product.

Clean the walking frame gently with water and commercial household cleaner. Then dry the product with a cloth.


So not use aggressive cleaners and always avoid exposing the walking frame to hard water jets (such as high pressure cleaners), otherwise small parts or the surface may be damaged.

 <b>WARNING!</b>	Warning, danger! Failure to comply with this hazard can result in death or serious injury
 <b>CAUTION!</b>	Caution, danger! Failure to comply with this hazard may result in minor or moderate injury
 <b>NOTE!</b>	Warning of possible damage to property


	Min. and max. height of the walking frame
	Max. width of the walking frame
	Max. user weight
	Can be used both inside and outside
	Observe user manual!
	Distributing company
	Date of manufacture
	CE conformity declaration
	Medical product
	Warning against hand injuries

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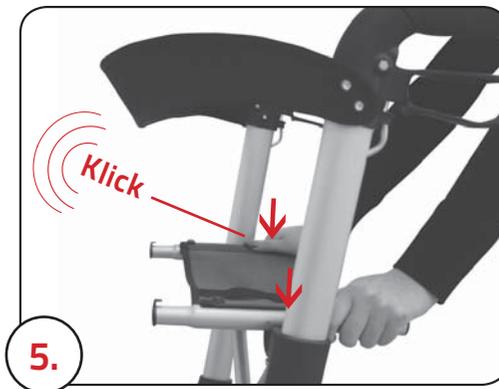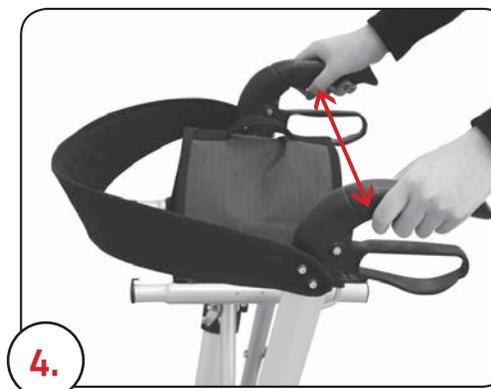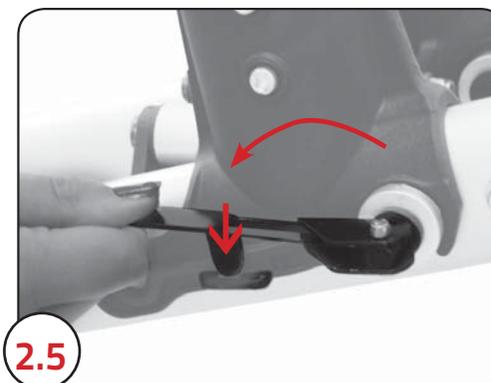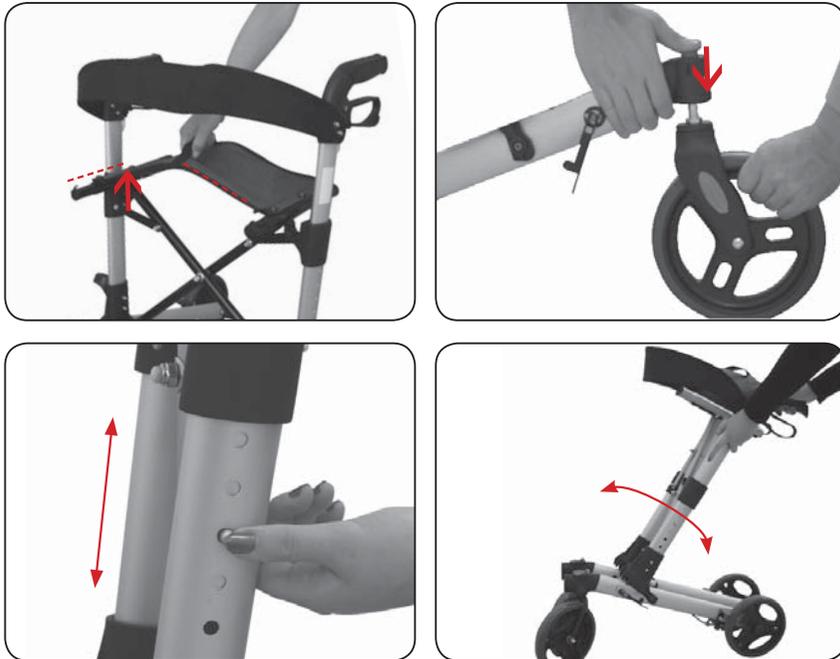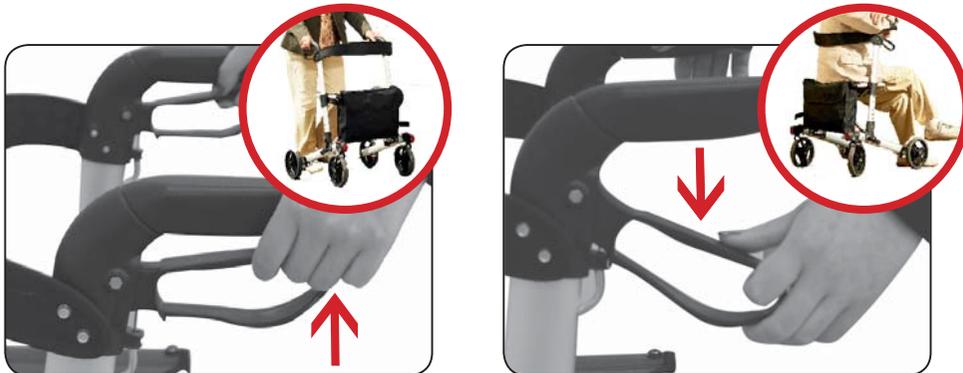Instruction Walking Frame A0300100

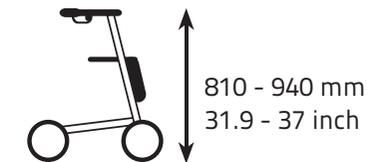
Dear customer,

thank you for choosing our product. Please read the safety instructions carefully before use and keep the manual for further use. The manufacturer assumes no liability for faulty assembly or improper use of the walking frame.

Delivery content

1x Pre-mounted Walking Frame

1x Shopping bag (Max. capacity: 13l, max. strain: 5kg/11 lbs)

1x user manual