RIDDER

Assistent

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**MONTAGEANLEITUNG
USER MANUAL**

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Gebrauchsanleitung Rollator A0300301 / A0300317

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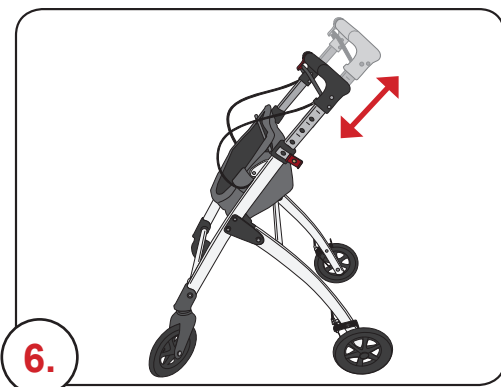
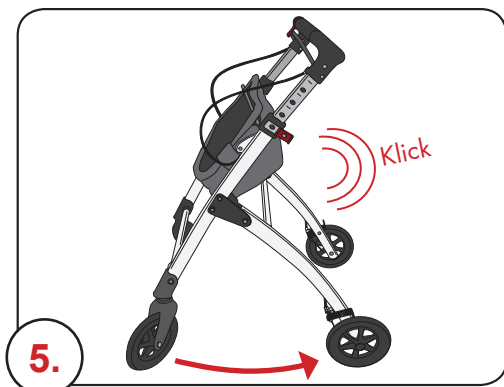
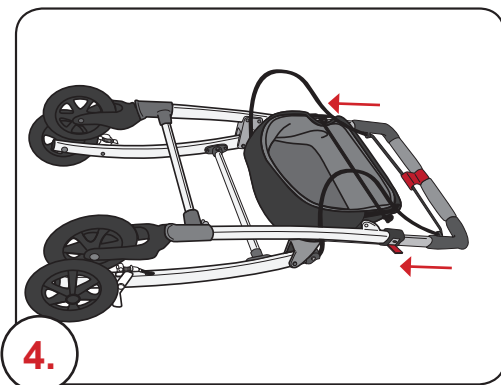
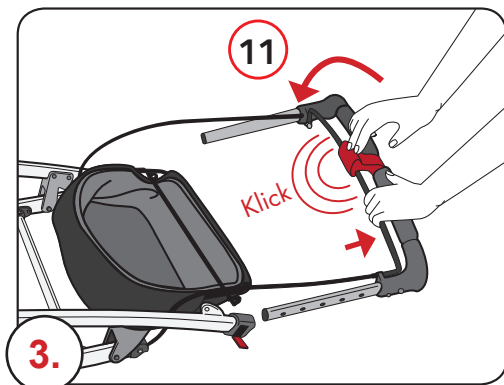
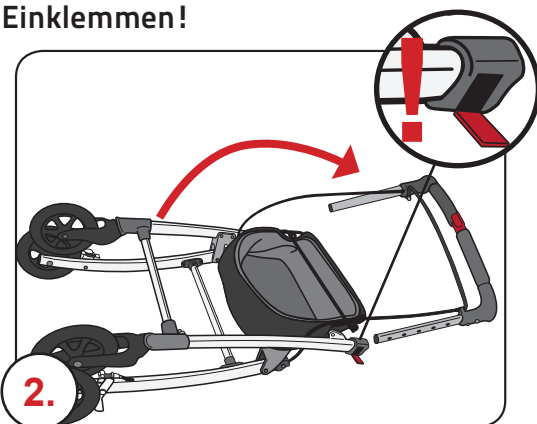
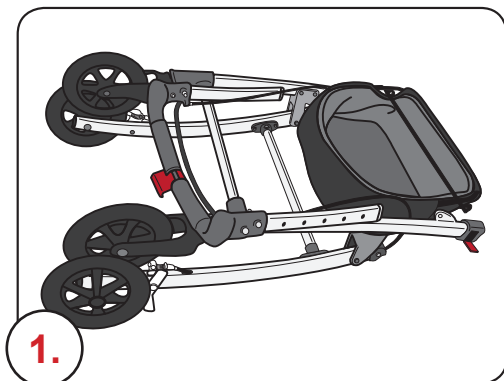
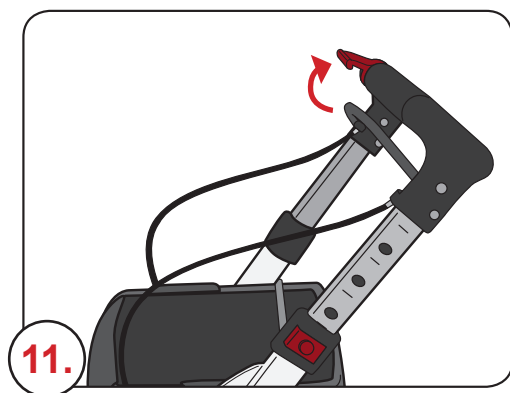
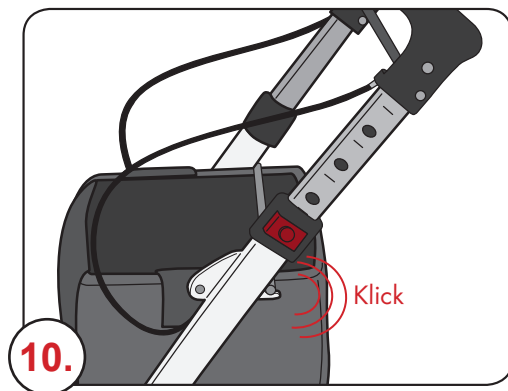
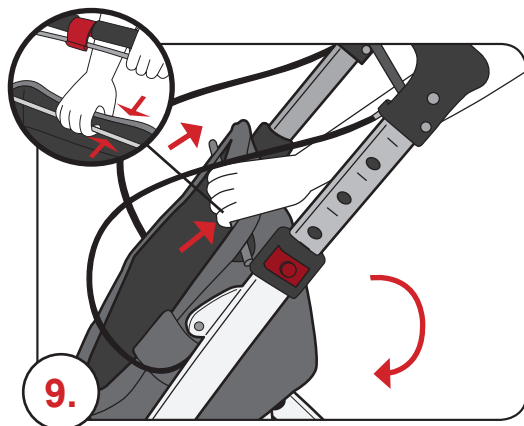
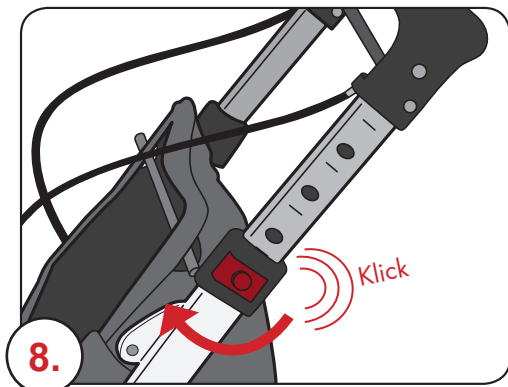
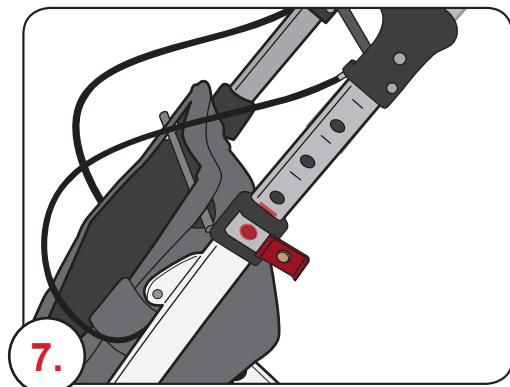
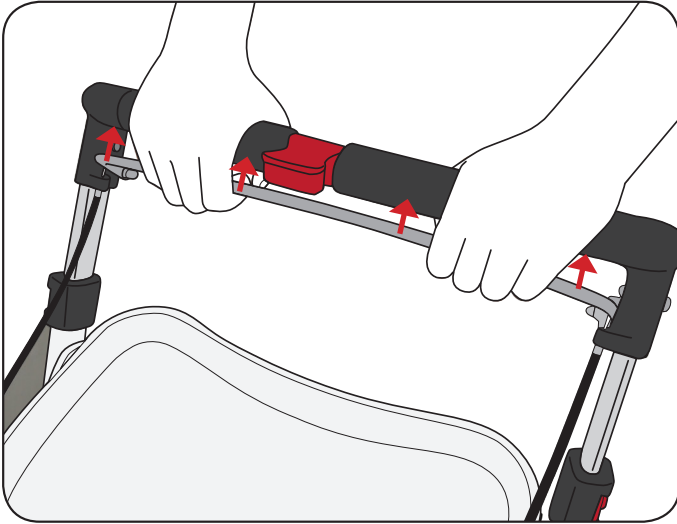
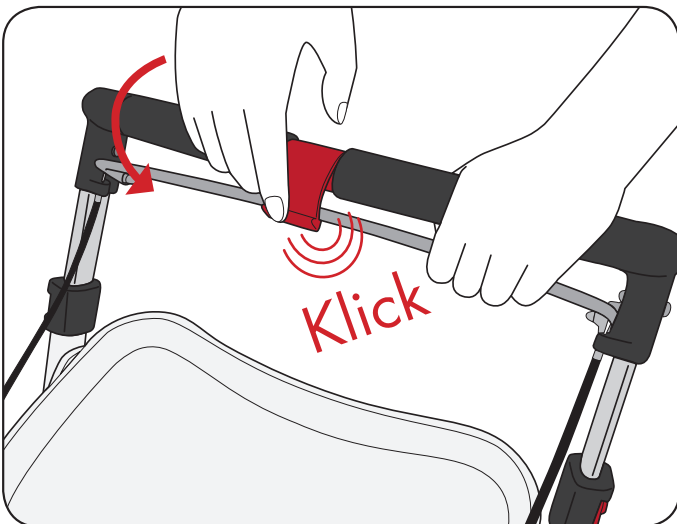
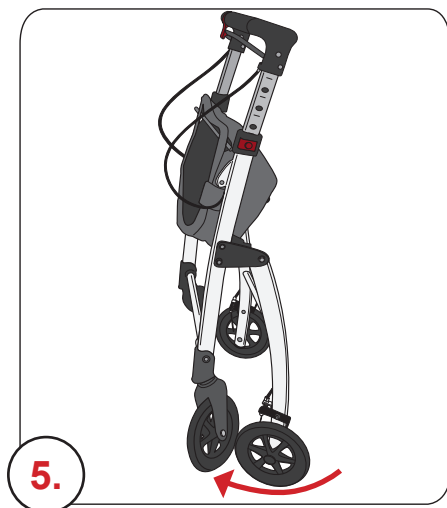
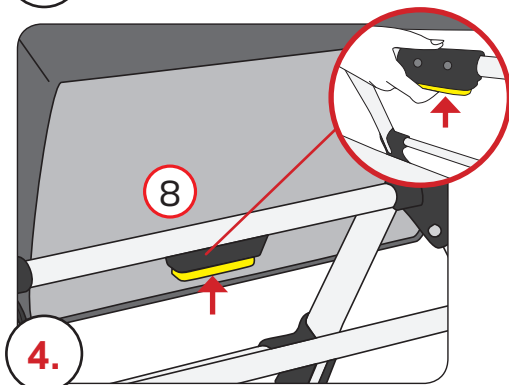
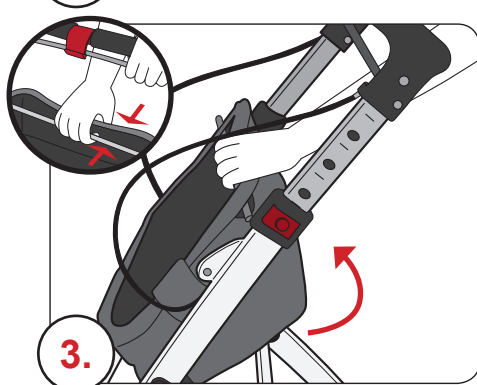
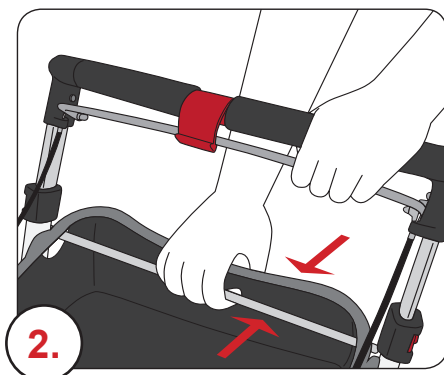
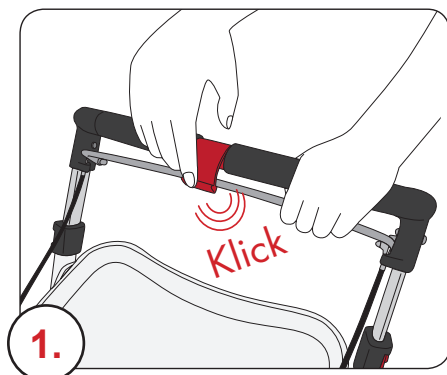
Instruction Walking Frame A0300301 / A0300317

Dear customer,

thank you for choosing our product. Please read the safety instructions carefully before use and keep the manual for further use. The manufacturer assumes no liability for faulty assembly or improper use of the walking frame.

Delivery content

1x Pre-mounted Walking Frame

835 - 930 mm

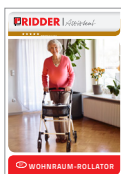
1x tablet

(Max. strain: 5kg/11lbs)

1x basket

(Max. capacity: 12 l, Max. strain: 5kg/11lbs)

1x instruction manual

 WARNING!	Warning, danger! Failure to comply with this hazard can result in death or serious injury
 CAUTION!	Caution, danger! Failure to comply with this hazard may result in minor or moderate injury
 NOTE!	Warning of possible damage to property

	Min. and max. height of the walking frame
	Max. width of the walking frame
	Max. user weight
	Can be used inside
	Observe user manual!
	Distributing company
	Date of manufacture
	CE conformity declaration
	Medical product
	Warning against hand injuries

NOTE!

Failure to follow these warnings can result in personal injury. DO NOT install or use this equipment before reading and understanding this instruction sheet. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional or technical personnel before attempting to install this equipment – otherwise, injure or damage may occur. You should always consult with your physician or therapist to determine the proper adjustment and usage.

The walking frame, which is only usable inside, was developed as a walking aid for motor-impaired humans. It can be used as seating as well.

The maximum recommended useful life is up to 5 years. In this specification the consideration of the intended use-, safety- and care instructions is compulsory prerequisite.

WARNING!

- Only use on flat and solid ground
- DO NOT sit on the walking frame
- Always check the brakes before each use and after every height adjustment in order to avoid free flex of the cable.
- Always check whether both of the handle tubes are secured at the same height and the snap buttons for the handgrip height adjust are fixed correctly.
- All wheels MUST be in contact with the floor at ALL TIMES during use.
This will ensure the walking frame is properly balanced.
- Keep packaging materials away from children.
- **Do not use in case of damage.**

NOTE!

Cleaning & Care:

Please follow the cleaning and care instructions. Improper use may damage the product.

Clean the walking frame gently with water and commercial household cleaner.
Then dry the product with a cloth.

So not use aggressive cleaners and always avoid exposing the walking frame to hard water jets (such as high pressure cleaners), otherwise small parts or the surface may be damaged.

i NOTE!

Important: Check all parts of shipping damage. In case of shipping damage, DO NOT use the product, contact the distributor for further instructions.

i NOTE!

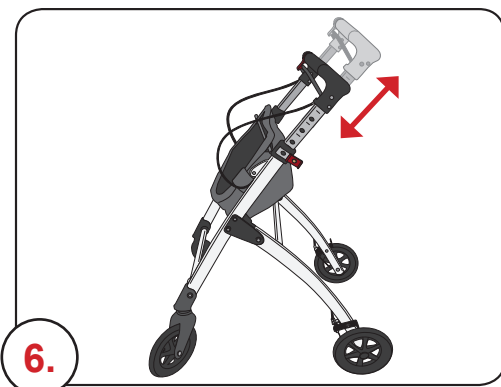
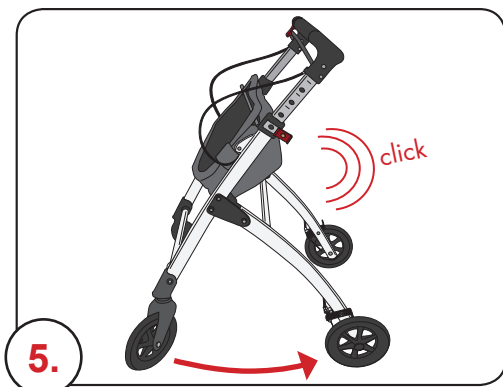
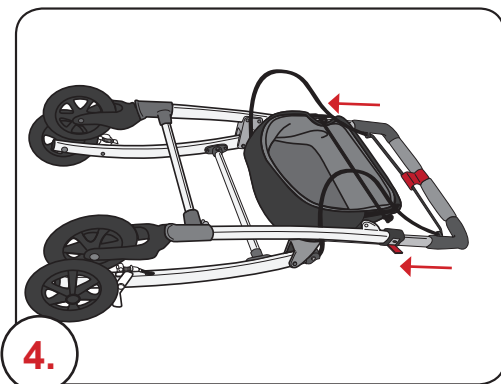
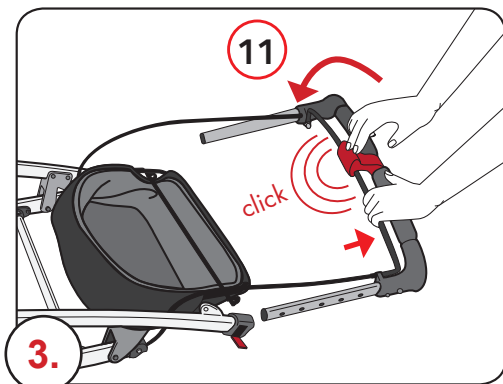
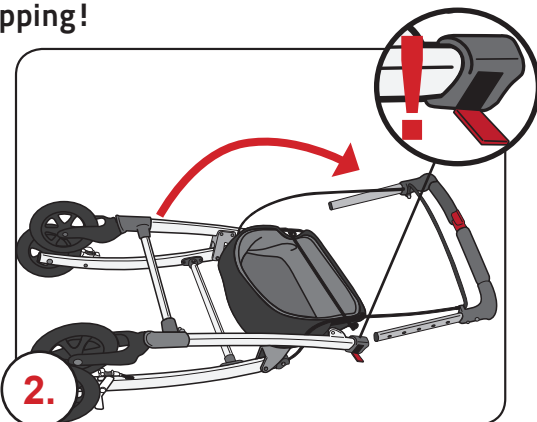
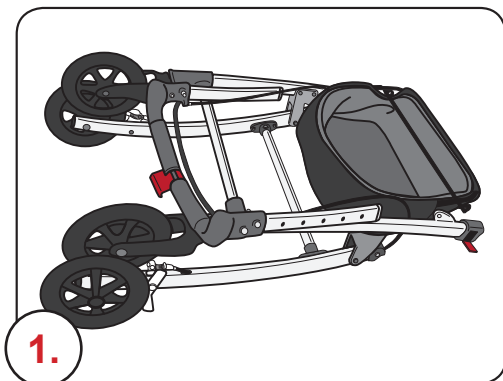
Stability warnings:

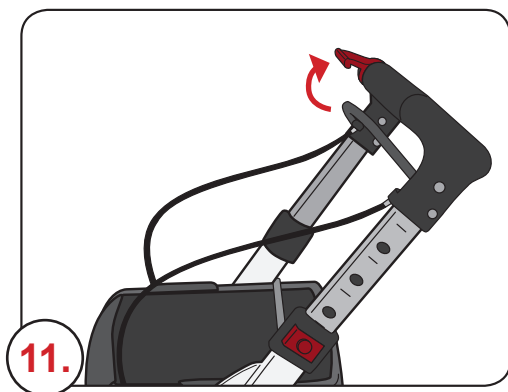
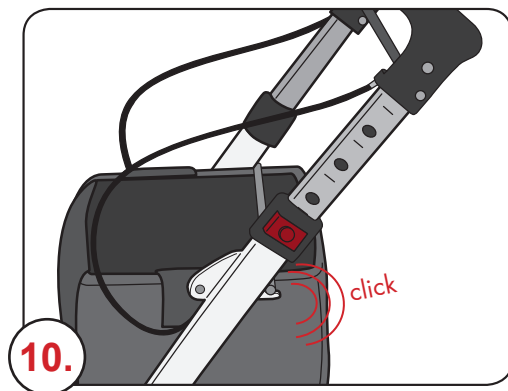
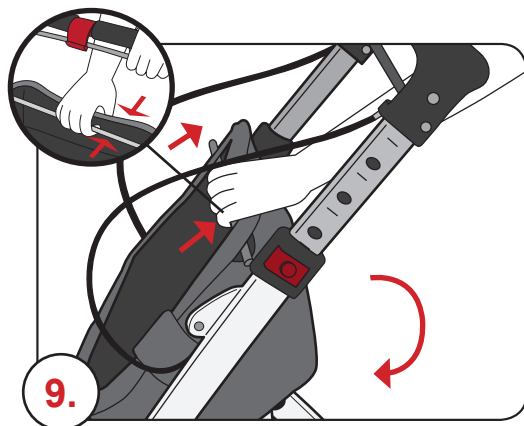
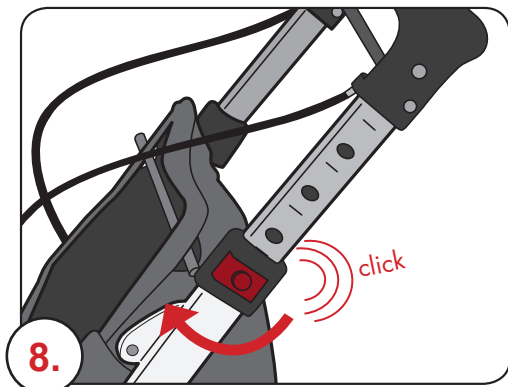
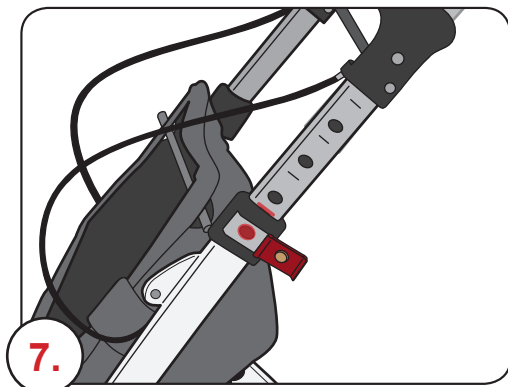
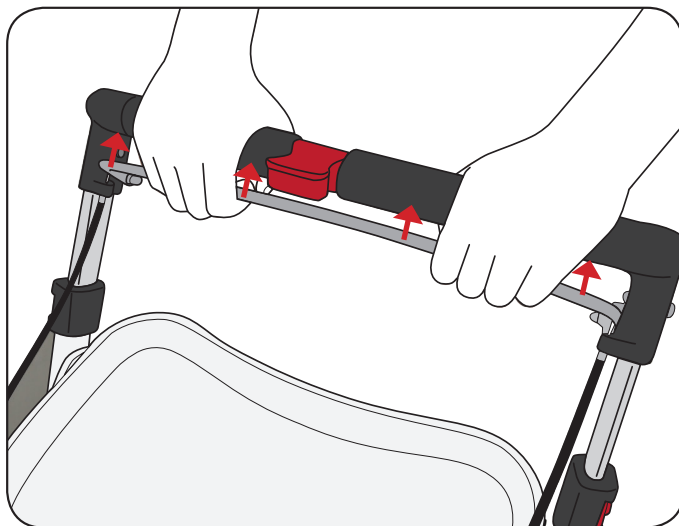
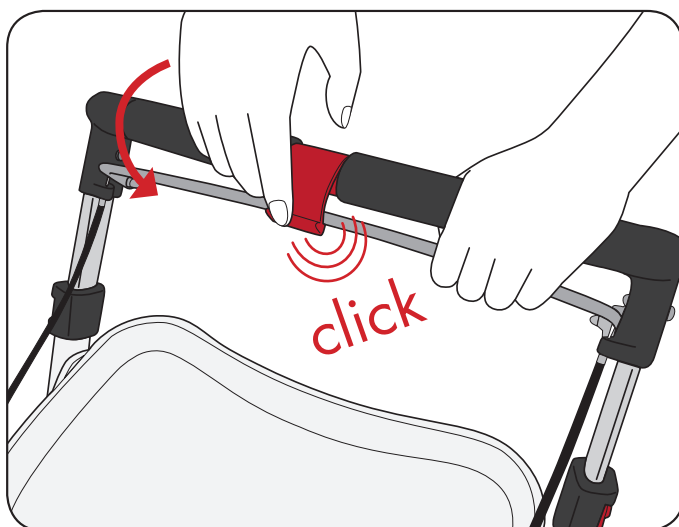
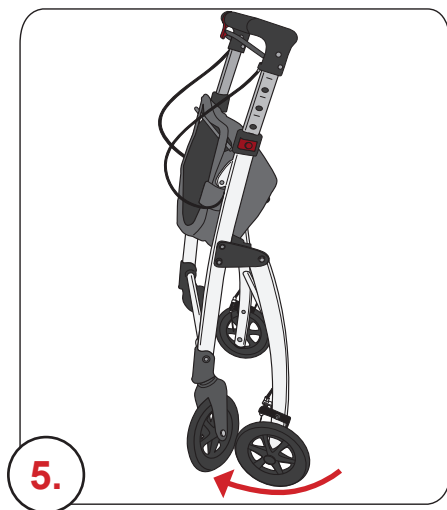
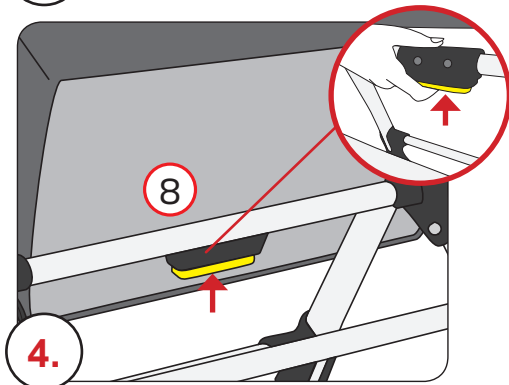
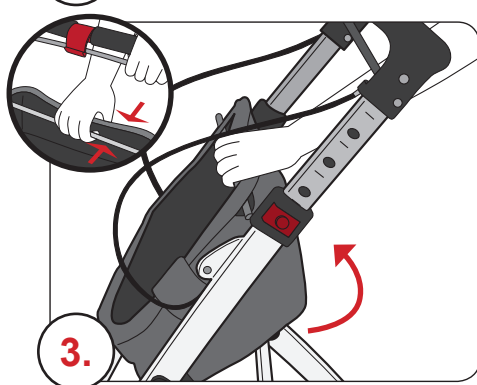
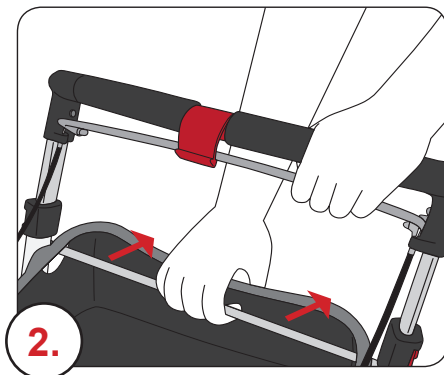
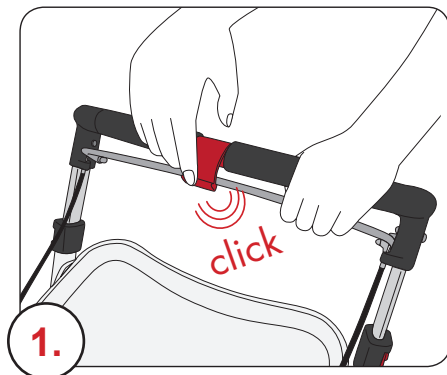
The walking frame can provide ambulatory assistance to an individual weighting up to 100kg (220 lbs). The basket has a weight limitation of 5kg (11 lbs).

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RIDDER GmbH